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# THROAT SPRAY

Gently soothes and moistens the throat and pharynx.

## What is newmed Throat Spray and how do you use the product?

Dry air (from air conditioning, overheated rooms, aircraft flights) and tiny foreign particles in the air (dust, pollen) can dry out the throat and irritate it. This can cause the throat's natural defences to be heavily impaired, and susceptible to infection by various germs. The mild, isotonic, preservative-free solution used in newmed Throat Spray can help here. newmed Throat Spray contains natural spring water from Switzerland, supplemented by plant extracts of sage, peppermint and edelweiss, to allow the throat to be moistened gently and naturally, thus preventing irritation and hoarseness.

newmed Throat Spray is used:

- to gently moisten the mucous membranes of the throat and pharynx
- to support treatment of sore throat and hoarseness (e.g. for colds)
- to protect against dryness in the throat area
- to support the throat's natural function
- to loosen mucus

## What do you need to note about using newmed Throat Spray?

- It is important to observe the User Information.
- If necessary, newmed Throat Spray can also be used over a longer period of time. Seek advice from medical staff, such as your doctor or pharmacist, if symptoms have not improved after 14 days.

## When must newmed Throat Spray not be used?

No unwanted effects are known when newmed Throat Spray is used properly. newmed Throat Spray should not be used in the case of known over-sensitivity to one or more of the ingredients.

If symptoms are persistent or recur frequently, seek advice from medical staff.

## When should you take special care when using newmed Throat Spray?

- No special precautions are necessary if used properly.
- If you are suffering symptoms following an ENT operation, you should seek advice from a doctor.
- Let medical staff know if you are suffering from any illnesses, have any allergies, or are taking other medicines. This includes medicines that you have bought yourself.

## How do you apply newmed Throat Spray?

For adults and children of 2 years of age and over, spray 1-2 jets into the throat several times a day, as required.

## Instructions for use:

1. Shake newmed Throat Spray before use. Before using the spray for the first time, actuate the atomizer mechanism several times until a fine spray vapour appears.
2. Direct the spray head into the throat area and spray once or twice. Hold the bottle as upright as possible whilst doing so.

For reasons of hygiene and to avoid infections, it is recommended that you wipe the spray adapter with a dry cloth after use, and close it again with the protective cap. Furthermore, it should be used by only one person.

## What side effects can newmed Throat Spray have?

To date, no side effects have been observed when used properly.

## What else must be noted?

- Store this product out of reach of children!
- Protect it from heat! Store at room temperature.
- Not for consumption!
- The preparation may be used only up until the expiry date (EXP) stated on the packaging or the container. After opening, newmed Throat Spray should be used within 6 months.

## What does newmed Throat Spray contain?

AQUA, LEONTOPODIUM ALPINUM (AQUA, GLYCERIN), XYLITOL, SALVIA OFFICINALIS, MENTHA PIPERITA, GLYCERIN, HYDROCHLORIC ACID

## Manufacturer:

New Care AG, CH-4452 Itingen

EU Office: Kreillerstr. 65, 81673 Muenchen / Germany

CE Class I medical device in accordance with the Medical Devices Directive 93/42/EEC and 2007/47/EC.

User information version: December 2010

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